


How to Use:

Recommended for ostomates using a drainable pouch, (not suitable for urostomates).

1. Fasten the drainable end of the pouch/ bag first.
2. Insert the ENTIRE sachet into the pouch from the top - DO NOT empty the contents into the pouch

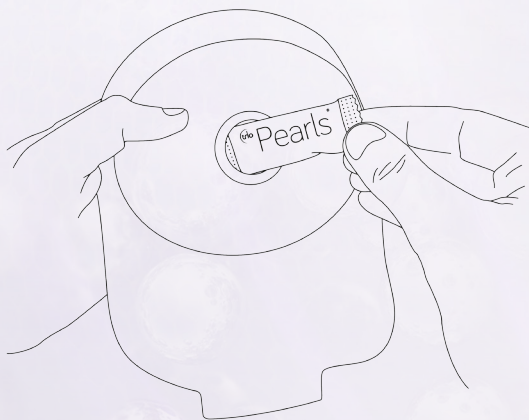

3. Further sachets can be used once you empty by inserting a sachet through the drainage outlet



Available now
Ordering information

100 sachets

REF TR205



Try it **FREE** now!

For a **FREE** sample please contact:

trio ostomy care™

TRIO HEALTHCARE LTD . MISSENDEN ABBEY . LONDON ROAD
GREAT MISSENDEN . BUCKINGHAMSHIRE . HP16 0BD

T: 0800 531 6280 E: mail@triohealthcare.co.uk

TrioOstomyCare.com

Follow us on



@TrioOstomyCare



Trio Ostomy Care

Order your **FREE** sample at: TrioOstomyCare.com

trusted results in ostomy

M10074

trio ostomy care™



trio Pearls®

super absorbent - gelling & odour
control sachets

The only **colourless** gel sachets with
active odour control!

trusted results in ostomy

Trio Pearls®

Trio Pearls® is a super absorbent gelling sachet that actively absorbs odour. Designed for ostomates with liquid stool, it can allow you to gain more control over the management of your ostomy and your life.

Liquid stool can cause embarrassment or discomfort by leaking, smelling, or feeling like it is 'sloshing' about. You may sometimes feel anxious or apprehensive to take part in social activities or anything where you feel you may not have access to a toilet for a long period of time.

Regular gelling agents can absorb the liquid in stool, but do not address the odour issue associated with it.

Trio Pearls® has NanoActive Technology, a nano-engineered ingredient that absorbs odorous molecules rather than just trapping them in a gel. Trio Pearls® can also provide control by absorbing the fluid in liquid stool, and gelling inside the pouch. The formed gel is thicker in consistency which helps to prevent leaks normally associated with liquid output; prevents the filter from clogging prolonging its efficiency; and avoids bulking by allowing the content to be flattened.

“Trio Pearls® doesn't stain the toilet and I noticed it didn't smell as much when I drained...”

Chris Jackson



Benefits:

- Actively absorbs odour during wear
- Preserves filter
- Prevents leakage
- Makes the pouch more discreet
- Easier drainage/less messy
- Gel does not stain clothes or toilet
- Security over longer periods
- Noticeable reduction in odour when draining
- Eliminates 'sloshing'

Perfect for:

- Overnight use for a good night's sleep
- Long journeys or flights – take it on holiday
- Social occasions such as parties, restaurants, theatre/cinema, etc.
- Active pursuits
- Everyday use
- Travel - the Travel pouch supplied makes Trio Pearls® portable on trips away

NanoActive Technology

Active Odour Control



The breakdown of foods in the digestion process produces odorous molecules and in turn, gases.

The new Trio Pearls® incorporate a superabsorbent gelling polymer and nano-engineered ingredients that actively absorb odorous organic molecules rather than just trapping them in the gel.

Fig 1. **Porosity** – How does NanoActive Absorb Odour?


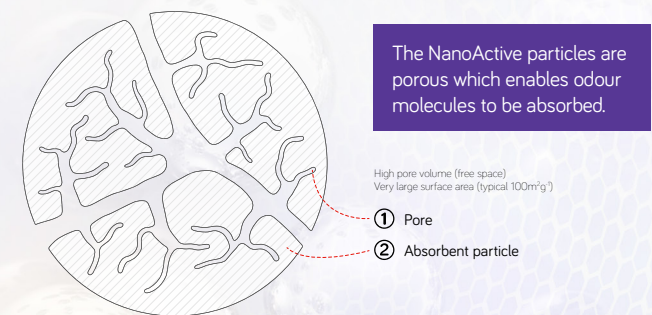


Fig 2. **Absorption** – The difference in odour control between Trio Pearls® and other Gelling products

Trio Pearls®

Competitor

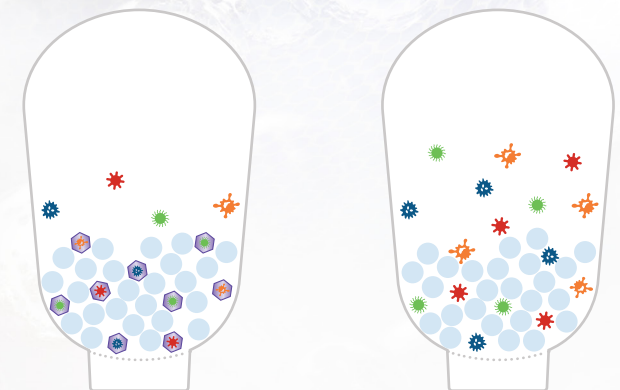

NanoActive Technology locks the smells away, so even when you drain you will notice significantly less odour.

Other gelling products in the market simply trap some of the smells in the gel rather than reducing it.

– NanoActive Technology

– Odour particles