




01 To open the tube, unscrew the cap. Puncture the foil with the reverse side of the cap.

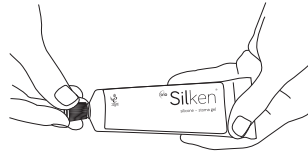

02 Clean area with water and ensure that the skin is dry.

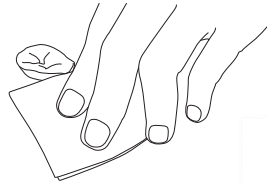

03 Squeeze the gel to fill dips and creases where required. For best results do not spread the gel.

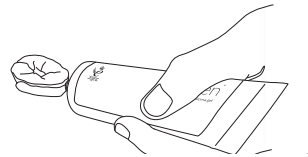

04 Apply your ostomy wafer or pouching system directly onto the gel and allow it to spread naturally. The gel will set underneath the wafer/pouching system during normal wear time.



Trio Silken® The advanced, soft silicone stoma gel:

- Fills skin folds, creases and uneven areas around the stoma
- Will not sting on application
- Allows the skin to breathe
- One piece removal - leaves no residue*
- Remains soft and flexible to ensure comfort

To find out more or to receive further samples:
please call us on **877-400-1700**
email us at **mail@trioostomycare.us**
or visit our website **www.TrioOstomyCare.us**



*when fully cured

To watch an instructional video on how to apply your Trio Silken® stoma gel just visit us at **www.TrioOstomyCare.us**

Trioostomycare USA 8235 Championsgate Blvd. Suite 2, Davenport, Florida, 33896

T:877-400-1700 E:mail@trioostomycare.us W:www.TrioOstomyCare.us