




01 Clean area with water and ensure that the skin is dry. (Attention: The seal will not stick to wet skin)

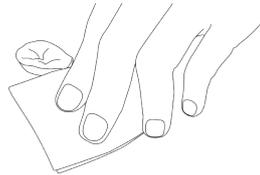

02 Remove by gently rubbing the edge of the seal toward the centre, then using thumb and forefinger carefully but firmly pull from the blister pack. The seal will normally be adhered to the blister.

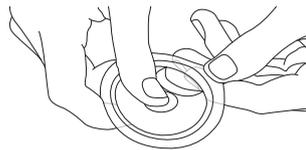

03 Stretch the Trio Siltac® or Trio Silvex® over the stoma and allow it to retract. Once the ring has retracted around the stoma, secure to the body by gently pressing on the seal until secure and comfortable.

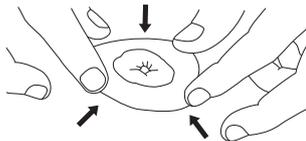

04 Place your ostomy pouching system directly onto the surface of the seal and hold in place with gentle pressure until secure.



Trio Silvex® The advanced, soft silicone convex seal.
Trio Siltac® The advanced, soft silicone ostomy seal.

- Allows the skin to breathe, reducing risk of skin irritation
- Reduced risk of leakage – extends wear time of your ostomy appliance
- Instant adhesion – better seal is formed
- Promotes comfort due to high level of flexibility
- More hygienic than traditional hydrocolloid products as it does not absorb faeces or urine

To find out more or to receive further samples,
please call us on **877-400-1700**
email us at mail@trioostomycare.us
or visit our website www.TrioOstomyCare.us



To watch an instructional video on how to apply your Trio Siltac® or Trio Silvex® seal just visit us at www.TrioOstomyCare.us

Trioostomycare USA 8235 Championsgate Blvd. Suite 2, Davenport, Florida, 33896

T: 877-400-1700 E: mail@trioostomycare.us W: www.TrioOstomyCare.us