




**01** Clean area with water and ensure that the skin is dry. (Attention: The flange extender will not stick to wet skin)

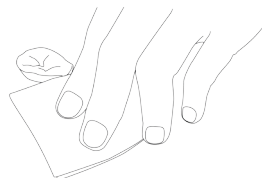


**02** Remove the clear release liner (Symbol No. 1) as indicated.

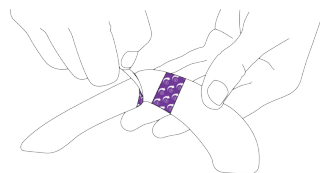

**03** Position where appropriate on your appliance and secure in place.

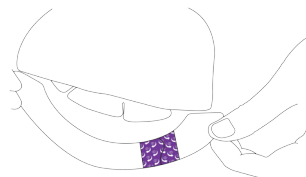

**04** Remove the paper release liner (Symbol No.2) as indicated and smooth down.

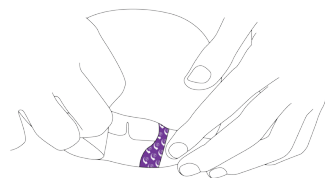

Trio Silex® The advanced, soft silicone gel flange extenders:

- Responsive Silicone™ flange extender
- Ultra thin, like second skin with instant adhesion
- Waterproof and non-absorbent, perfect for swimming and bathing
- Allows skin to breath, reducing risk of skin irritation
- Removes in one piece leaving no residue

To find out more or to receive further samples:  
please call us on 877-400-1700  
email us at [mail@trioostomycare.us](mailto:mail@trioostomycare.us)  
or visit our website [www.TrioOstomyCare.us](http://www.TrioOstomyCare.us)



To watch an instructional video on how to apply your Trio Silex® Flange Extender just visit us at [www.TrioOstomyCare.us](http://www.TrioOstomyCare.us)

Trio OstomyCare USA . 8235 Championsgate Blvd. Suite 2, Davenport, Florida, 33896

T:877-400-1700 E:[mail@trioostomycare.us](mailto:mail@trioostomycare.us) W:[www.TrioOstomyCare.us](http://www.TrioOstomyCare.us)