



**Elisse® Sting Free Skin Barrier**  
**say hello to**  
**healthier skin**

## how to use:

Elisse® Sting Free Skin Barrier uses bag-in-can technology – a recyclable and sustainable packaging system. Free from pollutants used in traditional aerosols, this packaging approach ensures that 100% of the product is used.

1. After removing your ostomy wafer, ensure that your skin is clean and dry.
2. Hold the can approximately 4-5 inches (10cm) away from skin and spray around your stoma where the adhesive wafer will go.
3. Elisse® can be used to build up layers of protection if required.
4. Allow the skin to dry - this should only take a few seconds. Then place your pouch onto your skin and secure.



### Also available in wipes

Handy size and individually wrapped for when you are traveling or on the go





**do regular pouch changes leave your skin constantly red and sore?**

**Elisse® Sting Free Skin Barrier builds a protective barrier between your ostomy wafer and your skin.**



**Prevents skin damage**

By stopping output from coming in contact with healthy or damaged skin



**Protects “at risk” peristomal skin**

From the strain of regular removals by building a barrier between skin and adhesive flange



**Allows skin to breathe naturally**

During wear, leading to better skin health



**Soothes irritated skin**

Our unique silicone formula calms red and itchy peristomal skin



**360° directional spray**

Sprays at any angle



**Dries in seconds**

Fast-drying, long-lasting protection

**Elisse® Sting Free Skin Barrier**  
**is available to purchase**  
**on our website at**  
**TrioOstomyCare.com**

Elisse® Sting Free Skin Barrier Spray		
Item number	Qty	HCPCS
TR204	1 can, 50ml (1.69oz)	A4369
Elisse® Sting Free Skin Barrier Wipes		
Item number	Qty/box	HCPCS
TR203	30 wipes	A5120



**TrioOstomyCare.com**

**877-694-3644**

**mygenii@TrioOstomyCare.com**