



Elite® Sting Free Adhesive Remover

say goodbye
to painful
removals

how to use:

Elite® String Free Adhesive Remover uses bag-in-can technology – a recyclable and sustainable packaging system. Free from pollutants used in traditional aerosols, this packaging approach ensures that 100% of the product is used.

1. Hold the spray can approximately 4-5 inches (10cm) away from the skin and spray at the top of the adhesive wafer.
2. Allow the adhesive remover to take effect. You should be able to feel the skin separate from the adhesive.
3. Peel away adhesive wafer from the skin and remove. Continue to spray to aid removal if required.
4. If you are left with any residue on your skin from your adhesive wafer, spray adhesive remover and gently wipe to remove. Alternatively, you can use Elite® wipes.



Also available in wipes

Handy size and individually wrapped for when you are traveling or on the go





does your adhesive remover sting when you spray it onto your skin?

**Elite® Sting Free Adhesive Remover gets to work on
the tough adhesives for fast and pain-free removal.**



Quick and easy to apply

For fast and pain-free
ostomy wafer removal



Allows skin to breathe

Doesn't leave any residue
or wetness on the skin



Kind to skin

Our unique silicone formula
is sting free and gentler on skin



Prevents skin damage

Reduces skin stripping caused
by repeated pouch changes



Dries in seconds

Fast drying




360° directional spray


Sprays at any angle

Elite® Sting Free Adhesive Remover
is available to purchase
on our website at
TrioOstomyCare.com

Elite® Sting Free Adhesive Remover Spray		
Item number	Qty	HCPCS
TR201	1 can, 50ml (1.69oz)	A4455
Elite® Sting Free Adhesive Remover Wipes		
Item number	Qty/box	HCPCS
TR202	30 wipes	A4456



 TrioOstomyCare.com

 877-694-3644

 mygenii@TrioOstomyCare.com