

**Pearls® Gelling &  
Odor Control Sachets**  
**say hello to  
confidence**

# Pearls® NanoActive Technology



Pearls® combine super absorbent gelling crystals and nano-engineered ingredients that actively absorb odorous organic molecules - providing greater security when you don't have easy access to empty or change your pouch.

## Gain confidence in all aspects of your life

- Use overnight for a restful sleep
- Take them on vacations or long trips
- Use every day for more control over your stoma
- Keep Pearls® handy for peace of mind at special social events

## how to use:

1. Simply pop the Pearls® sachet inside your new pouch. You don't need to empty the sachet.
2. Empty output as normal into the toilet or if you are using a closed bag, simply dispose of it as usual.
3. You can continue to add sachets after you have emptied your output by inserting another sachet through the drainable outlet.

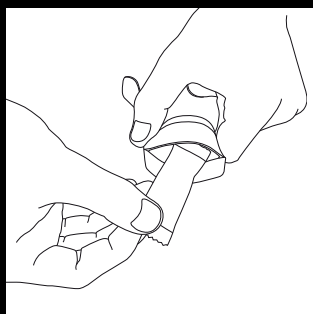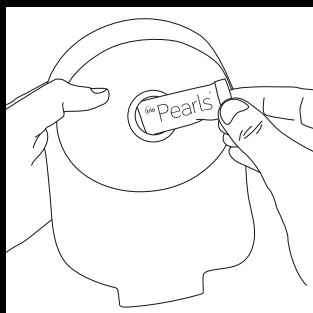

Pearls® mean less  
stress and less mess





**are you worried about leaking,  
odor, or output ‘sloshing’ around  
in your pouch?**

**Pearls® are super absorbent  
gelling and odor control sachets.**



**Prevents leakage**

By solidifying  
liquid output



**Controls odors**

Actively absorbs odorous  
molecules and gases




**Colorless**


Doesn't stain  
clothes or toilet

Pearls® Gelling & Odor Control Sachets  
are available to purchase  
on our website at  
TrioOstomyCare.com

| Pearls® Gelling & Odor Control Sachets |             |       |
|--|-------------|-------|
| Item number                            | Qty         | HCPCS |
| TR205                                  | 100 sachets | A4422 |



 [TrioOstomyCare.com](https://TrioOstomyCare.com)

 877-694-3644

 [mygenii@TrioOstomyCare.com](mailto:mygenii@TrioOstomyCare.com)