



Silex® Silicone Ostomy Flange Extenders

say YES
to extra
security

trio ostomy care™



why is **Sil2** Breathable Silicone Technology® great for skin?




Sil2 Breathable Silicone Technology® is a unique silicone adhesive blend that feels and breathes just like normal skin. It sticks exceptionally well but doesn't absorb moisture. So it won't swell, become mushy, create a gooey mess, or cause pain when you remove it - which means you can wear it for longer without worrying about leaks.



how to use:

It's important that you do not remove all the release liners before applying, and follow the step 1 and step 2 guidance to ensure easy application and secure adhesion to the skin and wafer.

1. Apply your pouch as usual. It's important to make sure the skin around your wafer is dry as **the flange extender won't stick to wet skin.**
2. Remove the clear release liner, marked 1, from the flange extender.
3. Position the flange extender securely on the skin and around the wafer.
4. Remove the paper release liner, marked 2, and smooth it down.
5. You can do the same with a second extender. Flange extenders can be placed in different positions to get the best results.

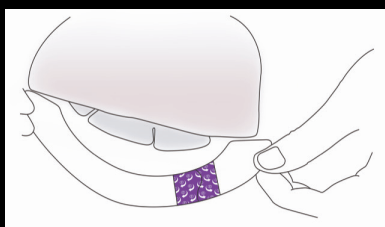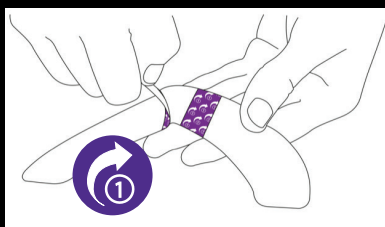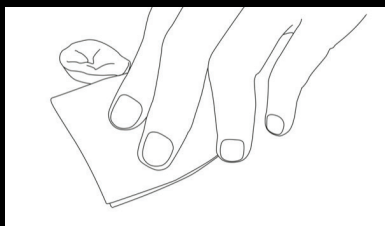



**extend the security of your ostomy pouch
with soft silicone extenders.**

**Silex® Silicone Ostomy Flange Extenders are
designed to go around the wafer to provide a
soft, flexible, invisible border that conforms
to your body shape for extra security.**



Secure adhesion

Stays put no matter
the activity, body shape,
or temperature



Comfortable

Ultra thin and invisible -
like a second skin



Waterproof

Ideal for bathing
and swimming



Highly flexible

Helps you move
freely and easily



Gentle

Pain-free removal and
no residue left behind



Longer wear time

Increases the wear
time of your pouch



Silex® Silicone Ostomy Flange Extenders

Our Flange Extenders are part of a complete range of Trio® solutions. Silex® Silicone Flange Extenders are available in one size which will fit around all pouch sizes and shapes.

Silex® Silicone Ostomy Flange Extenders

Item number	Size	Stoma size	Qty/box	HCPCS
TR1060	—	All	20	A4362

Siltac® Silicone Ostomy Seals

Item number	Size	Stoma size	Qty/box	HCPCS
TR1020-10	1	3/4" - 1 1/8" (20-28mm)	10	A4385
TR1028-10	2	1 1/8" - 1 3/8" (28-35mm)	10	A4385
TR1035-10	3	1 3/8" - 1 3/4" (35-44mm)	10	A4385

Silvex® Silicone Convex Ostomy Seals

Item number	Size	Stoma size	Qty/box	HCPCS
TR1020CX	1	3/4" - 1 1/4" (20-30mm)	10	A4411
TR1030CX	2	1 1/4" - 1 5/8" (30-40mm)	10	A4411




Are you getting leaks?

Does your wafer or seal become gooey and sticky close to the stoma? Is the skin under your wafer uneven, sensitive, red, or sore? Our **Ostomy Seals** are easy to apply. They simply stretch around your stoma, without the need for warming or shaping, and then retract to provide a custom fit. Get in touch today if you'd like more information or to order your free sample.



 TrioOstomyCare.com

 **877-694-3644**

 mygenii@TrioOstomyCare.com