



# Silken® Silicone Ostomy Gel

# say goodbye to leaks

**trio** ostomy care™






# why is **Sil2** Breathable Silicone Technology® great for skin?

Sil2 Breathable Silicone Technology® is a unique silicone adhesive blend that feels and breathes just like normal skin. It sticks exceptionally well but doesn't absorb moisture. So it won't swell, become mushy, create a gooey mess, or cause pain when you remove it - which means you can wear it for longer without worrying about leaks.



## how to use:

1. Prepare the area around your stoma, ensuring that the skin is clean and dry.
2. To open, unscrew the cap and puncture the foil with the reverse side of the cap.
3. Gently squeeze the gel to fill dips and creases as required. Only a small amount of gel is needed. Using too much gel creates a mess and decreases the adhesiveness of the wafer.
4. There's no need to spread the gel or wait until it's dry. Simply apply your pouch as usual, placing the wafer directly onto the moist gel. The gel will solidify slowly, remaining soft until the creases and gaps in your skin have been filled.

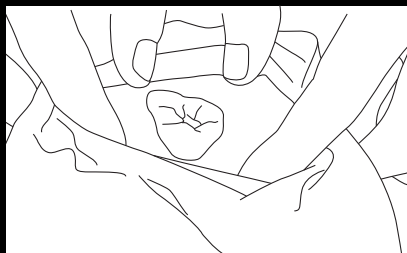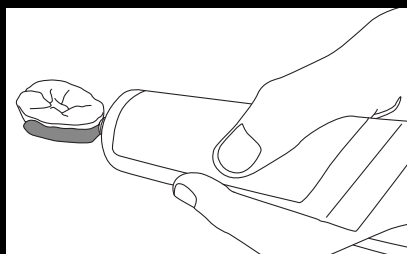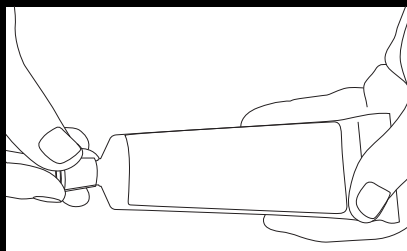



**is the skin surrounding your stoma uneven and causing leaks?**

**Silken® Silicone Ostomy Gel creates an even surface for optimal pouch application. Plus - it's sting free!**



**Allows skin to breathe**

Reduces skin damage and keeps skin healthy



**Flexible and secure**

Responds to body movement, filling the gaps caused by skin folds, dips, creases, and uneven skin



**Quick and easy**

Removes in one piece and doesn't leave residue on the skin




**Protects delicate skin around your stoma**


Prevents output from getting into contact with the skin


**Silken® Silicone Ostomy Gel**  
**is available to purchase**  
**on our website at**  
**TrioOstomyCare.com**

Silken® Silicone Ostomy Gel		
Item number	Qty	HCPCS
TR1070	1 tube, 60g	A4406



 **TrioOstomyCare.com**

 **877-694-3644**

 **mygenii@TrioOstomyCare.com**

Trio Healthcare Inc., 116 Village Blvd. Suite 200, Princeton, NJ 08540  
Silken and Sil2 are registered trademarks of Trio Healthcare Ltd. ©Trio Healthcare Ltd. 2022