




## Siltac® and Silvex® Silicone Ostomy Seals

here's a  
peek at  
preventing  
leaks...






# why is **Sil2 Breathable Silicone Technology®** great for skin?

Sil2 Breathable Silicone Technology® is a unique silicone adhesive blend that feels and breathes just like normal skin. It sticks exceptionally well but doesn't absorb moisture. So it won't swell, become mushy, create a gooey mess, or cause pain when you remove it - which means you can wear it for longer without worrying about leaks.

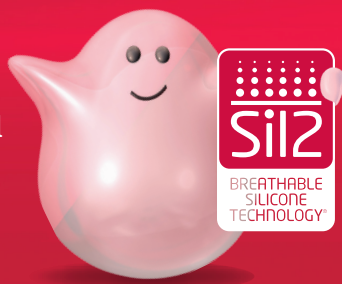

## how to use:

1. Clean the area around the stoma. It's important to make sure the skin is dry, as **the seal won't stick to wet skin.**
2. Using your thumb and forefinger, carefully pull the seal from its pack. It's normal for the seal to be fixed to the blister pack, but it's strong enough for you to pull and stretch it free without damage.
3. Take the edge of the seal in both hands and stretch it around the stoma. You can then allow the seal to retract around the stoma to form a comfortable seal.
4. The ostomy seal stretches to fit around the stoma. So there's no need to mold it into shape. Its gentle elasticity will hug the stoma to form the seal.
5. Once it has sealed around the stoma, gently press the seal onto your skin. Place the ostomy pouch directly onto the seal as usual, and hold it in place until it feels secure.

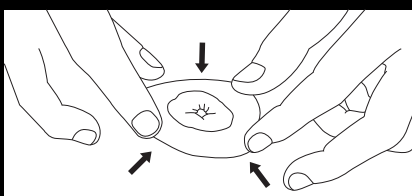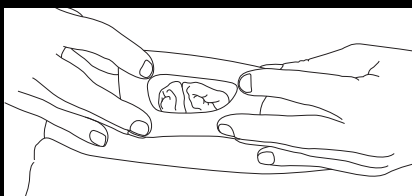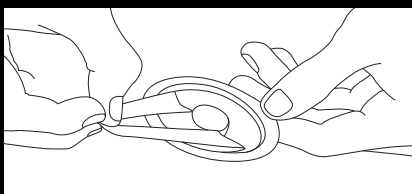

**CONVEX**



**CONCAVE**

**Convex seals** are versatile and can be applied in two ways, either convex side up or convex side down, so that gentle pressure can be applied directly around your stoma to create a secure fit, whichever works best for you.



## the first silicone ostomy seal that provides a perfect fit

**Siltac® and Silvex® Silicone Ostomy Seals are  
available in flat and convex to suit all stoma types.**



### **Snug, customized fit**

Hugs the stoma, eliminating  
gaps and leaks



### **Comfortable and secure**

Responds to body movement



### **Quick and easy to apply**

Simply stretch around the stoma  
without warming or shaping



### **Protects delicate skin around your stoma**

Protection from stool and  
urine, allowing skin to heal  
naturally and remain healthy



### **Longer wear time**

Unique Sil2 Breathable Silicone  
Technology® doesn't absorb  
bodily fluids so it won't become  
goopy, mushy, or break down

**Also available! Silvex®  
Convex Ostomy Seals**

Ideal for flat or inverted stomas or uneven skin





## Siltac® and Silvex® Silicone Ostomy Seals

Our flat and convex seals are part of a complete range of Trio® solutions. Your stoma may change shape and size but our seals will flex and move with your stoma. Find the best one for you by selecting your stoma size and choosing 1, 2, or 3.

### Siltac® Silicone Ostomy Seals

Item number	Size	Stoma size	Qty/box	HCPCS
TR1020-10	1	¾" - 1 ⅛" (20-28mm)	10	A4385
TR1028-10	2	1 ⅛" - 1 ⅜" (28-35mm)	10	A4385
TR1035-10	3	1 ⅜" - 1 ¾" (35-44mm)	10	A4385

### Silvex® Silicone Convex Ostomy Seals

Item number	Size	Stoma size	Qty/box	HCPCS
TR1020CX	1	¾" - 1 ¼" (20-30mm)	10	A4411
TR1030CX	2	1 ¼" - 1 ⅝" (30-40mm)	10	A4411

### Silex® Silicone Ostomy Flange Extenders


Item number	Size	Stoma size	Qty/box	HCPCS
TR1060	—	All	20	A4362




### Need extra security?

You may have a hernia, do a lot of physical activity, or want peace of mind at night. Our **Flange Extenders** are designed to go around the flange to provide a soft, flexible, invisible border which conforms to your body shape to provide extra security. Get in touch today if you would like more information, or to order your free sample.



 [TrioOstomyCare.com](https://www.TrioOstomyCare.com)

 **877-694-3644**

 [mygenii@TrioOstomyCare.com](mailto:mygenii@TrioOstomyCare.com)